

=====

Radcal, Monrovia, CA (626)357-7921

Software Version 2.82Build 2 for use with Nugget firmware version 1.20 and up
For use with Radcal Accu-Gold series digitizers, the Accu-Gold Nugget and the Accu-Gold Touch

=====

Notes: (Change list follows on page 2)

Nugget - If you haven't done so already, please update the Nugget firmware as indicated above.

Accu-Gold version 2 is required for Nugget operation.

Before using the Nugget, read the Accu-Gold Manual, Appendix C - Nugget. The manual is accessible by choosing Help from the Accu-Gold 2 software. It can also be accessed from this USB Flash Drive in the Setup Content folder. The Nugget is delivered fully charged.

The Nugget can be used with non-plus legacy AGMS sensors. However, the calibration info has to be copied to the Nugget. We have copied any AGMS calibrations to your Nugget based on our information. If a different sensor is being used, please contact Customer Support and they will help you.

Both the version 1 and version 2 Accu-Gold software can exist on your computer. But only one can be used at a time to make measurements. See manual for more details.

AG2 Installation Notes:

1. **Install Accu-Gold software prior to connecting the Digitizer Module.**

Uninstalling old versions is not necessary.

3. Administrator privileges are required for installation.

Installation Procedure:

1. To begin Accu-Gold2 software installation, launch the file "Accu-Gold_Setup" located in the root of the flash drive.
2. The "Accu-Gold_Setup" file will launch an installation dialog. You must agree to the *License terms and conditions* to continue. Click **Install**. The dotNET Framework Client version 4.0 or greater is required and will be installed if it is not already installed on your computer. An internet connection is not required.
6. Click "Close" to exit the installation process.

Uninstall Procedure:

1. To uninstall Radcal Accu-Gold go to Add/Remove Programs in Control Panel and select "Accu-Gold 2". Then click on Uninstall and answer Yes to the prompts. You may also run the setup program again to uninstall it.

Accu-Gold Excel is also included on the flash drive but cannot be used with the Nugget. Refer to the Accu-Gold Excel readme file for installation.

Please feel free to comment on this software and report any bugs that you may find. Please save session files since they would be most useful. A license/calibration file is required for the AGMS multi-sensor. Please contact customer support if you do not have one for your multi-sensor. The AGMS+ multi-sensors have built in calibration.

For more in-depth information or troubleshooting,

Contact: Customer Support -
(626) 357-7921 x123
Cust_Sup@radcal.com