


AN1005 - Accu-Gold Registration


For AG2

- Launch Accu-Gold 2
- On the first screen, select "Unlock Now"

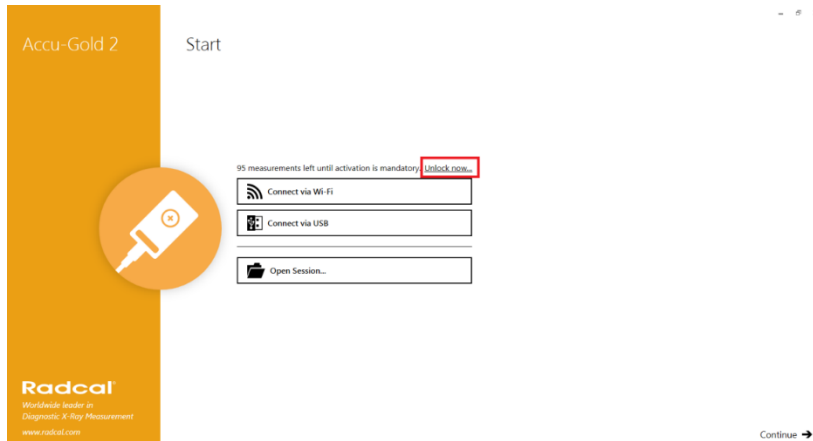

For AG3

- Launch Accu-Gold 3
- On the first screen, select "Settings"
- On the next screen, select "Help"
- Then select "Unlock now..." next to "Manage Activation"

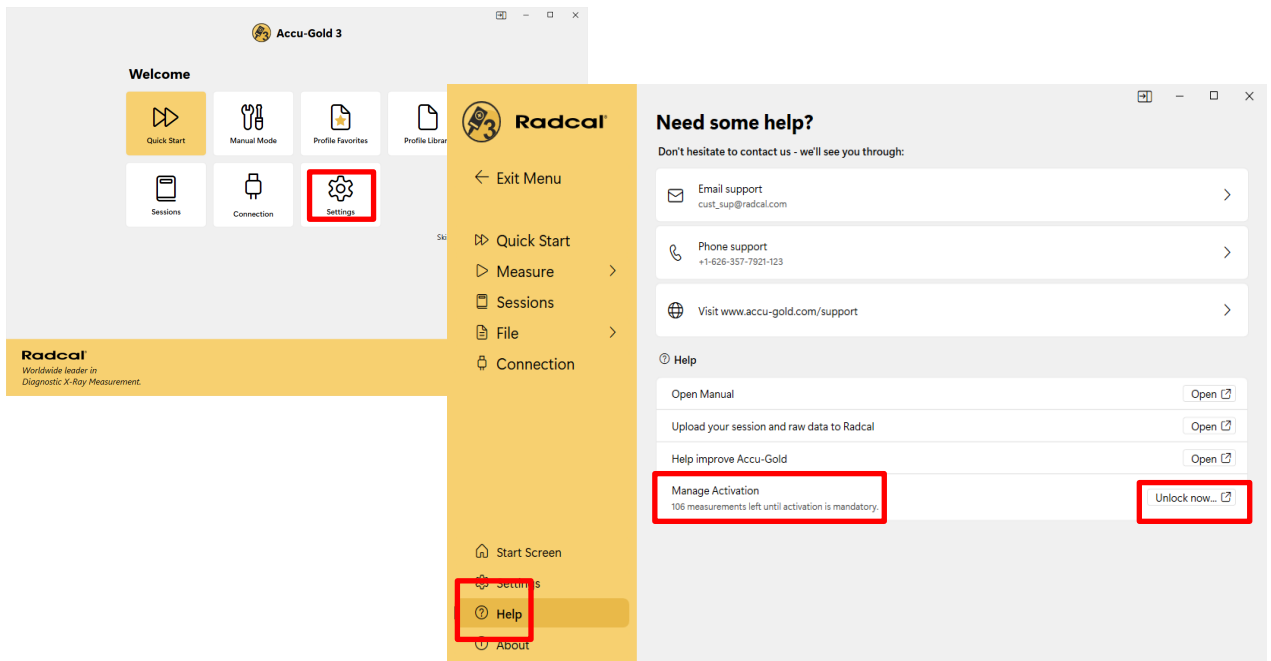

AN1005 - Accu-Gold Registration



- Then click “Get your free key now”

1. Start 2. AGDM 3. Registration

×



Get your free key now
to unlock unlimited measuring

If you already have an activation key by hand: [Activate with an activation key](#)

We are here for you. Don't hesitate to contact us: cust_sup@radcal.com

AN1005 - Accu-Gold Registration



- Connect the Digitizer module then click “Connect”

1. Start 2. **AGDM** 3. Registration

×



Please connect your digitizer module via USB now!
Registration will include your Accu-Gold digitizer serial number.

Connect

We are here for you. Don't hesitate to contact us: cust_sup@radcal.com

- Once the Digitizer is recognized click “Proceed”

1. Start 2. **AGDM** 3. Registration

×



Done.

Proceed

We are here for you. Don't hesitate to contact us: cust_sup@radcal.com

AN1005 - Accu-Gold 2 Registration

- Click "Get Key"
- If registering using another computer, clicking the arrow for offline activation. It will show the Computer ID. Write it down. **3. Registration**

Get free activation key
Click below to open the online registration form in your browser:
[Offline or having trouble? Activate manually...](#)

Please visit the following link and enter your computer id:

Computer ID:

Registration Link: <https://accu-gold.com/register>

Get Key

Using the ID, you can register using a different computer

Enter activation key
Enter the key you have received online or via email here:

Unlock

We are here for you. Don't hesitate to contact us: cust_sup@radcal.com

- The web browser will automatically open to the registration page.
- The Computer ID will automatically be entered. Fill out all other information then click "Register"

Accu-Gold Registration

Please register to get your activation key.

Name:

Job Position:

Email:

Organisation:

Country:

Application:

Computer ID:

I agree to Radcal's privacy policy and agree to receive sporadic information from Radcal.

Register

If you have any questions or do not wish to be registered, please get in touch!

Choose
Accu-Gold 2 or
Accu-Gold 3
as appropriate

Enter the
Computer ID
obtained from
the previous step
if necessary

AN1005 - Accu-Gold Registration



- Note the activation key then return to Accu-Gold 2 or Accu-Gold 3

Accu-Gold 2 Registration
Support Restart Registration

End, this is your activation key:

741914

Copy to clipboard

With this key you can enable unlimited measurement in Accu-Gold 2.

Open the Accu-Gold 2 menu and select "Help - Manage Activation". Click on "Activate" and enter your activation key.

You need to register every computer you want to use for measurement. Computers you use for viewing saved sessions don't need to be registered. Registration is free and ensures that we can send you information on important updates or new features. You can see our privacy policy [here](#).

Are you having a problem with the activation? Please get in touch: [Radcal Registration Support](#)

© Radcal Corporation - All rights reserved | [Privacy Policy](#)
v1.1

- Enter the activation key then click "Unlock"

1. Start
2. AGDM
3. Registration
x

Get free activation key

Click below to open the online registration form in your browser:

⌂ Offline or having trouble? Activate manually...

Please visit the following link and enter your computer id:

Computer ID:

Registration Link: <https://accu-gold.com/register>

Enter activation key

Enter the key you have received online or via email here:

We are here for you. Don't hesitate to contact us: cust_sup@radcal.com

AN1005 - Accu-Gold Registration

- The registration process has now been completed. Click “Close” and enjoy Accu-Gold.

1. Start 2. AGDM 3. Registration

Success!

Accu-Gold 2 is unlocked for unlimited measurements.



Close

We are here for you. Don't hesitate to contact us: cust_sup@radcal.com

Alternative Registration Options

Using the Computer ID obtained on page 3, you can:

- Go to a location that has WiFi to complete the registration process.
- or
- Email custsup@radcal.com with the Computer ID to receive the activation key.
- or
- Using a smartphone you can use the internet browser to access the registration page: <https://accu-gold.com/register>.
- You will need to manually enter the Computer ID to get the activation key. You then go back to the original computer and enter the activation key to complete the process.

For assistance contact cust_sup@radcal.com