


Accu-Gold 2 Registration

- Launch Accu-Gold 2
- Follow the On-screen prompts till you have reached the connection selection and select "Unlock Now"

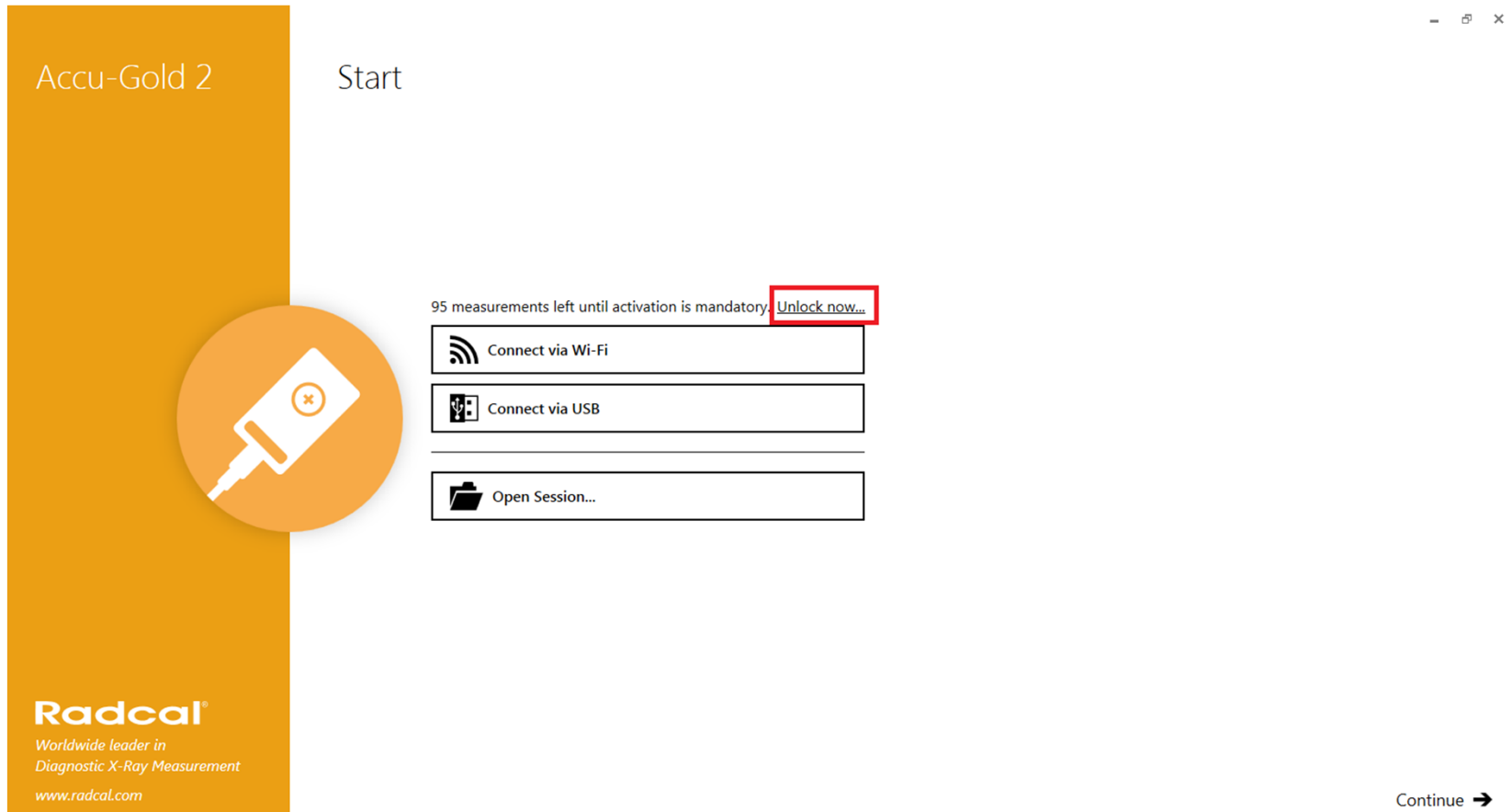

The screenshot shows the Accu-Gold 2 software interface. On the left, there is an orange vertical bar with the text "Accu-Gold 2" and the Radcal logo at the bottom. The logo includes the text "Radcal®", "Worldwide leader in Diagnostic X-Ray Measurement", and "www.radcal.com". In the center, the word "Start" is displayed. Below it, there is a message: "95 measurements left until activation is mandatory." followed by a red-bordered button labeled "Unlock now...". Below this message are three connection options: "Connect via Wi-Fi", "Connect via USB", and "Open Session...". Each option is accompanied by a small icon (Wi-Fi, USB, and a folder icon respectively). In the bottom right corner, there is a "Continue →" button.

-Click “Get your free key now”

1. Start 2. AGDM 3. Registration

×



If you already have an activation key by hand: [Activate with an activation key](#)

We are here for you. Don't hesitate to contact us: cust_sup@radcal.com

-Connect the Digitizer module then click “Connect”

1. Start 2. AGDM 3. Registration



Please connect your digitizer module via USB now!
Registration will include your Accu-Gold digitizer serial number.

Connect

We are here for you. Don't hesitate to contact us: cust_sup@radcal.com

-Once the Digitizer is recognized click “Proceed”

1. Start 2. AGDM 3. Registration



Done.

Proceed

We are here for you. Don't hesitate to contact us: cust_sup@radcal.com

-Click "Get Key"

-If registering using another computer, clicking the arrow for offline activation will show the Computer ID. Write it down.

1. Start 2. AGDM 3. Registration

Get free activation key

Click below to open the online registration form in your browser:

Offline or having trouble? Activate manually...

Please visit the following link and enter your computer id:

Computer ID: 561-909-189

Registration Link: <https://accu-gold.com/register>

Get Key

Using the ID, you can register using a different computer

Enter activation key

Enter the key you have received online or via email here:


XXXXXX

Unlock

We are here for you. Don't hesitate to contact us: cust_sup@radcal.com

- The web browser will automatically open to the registration page.
- The Computer ID will automatically be entered. Fill out all other information then click “Register”

Accu-Gold 2 Registration Support

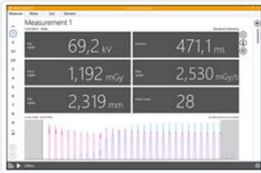

Accu-Gold 2 Registration

Please register to get your activation key for Accu-Gold 2.

Name:

Job Position:

Email:

Organisation:

Country:

Computer ID:

I agree to Radcal's privacy policy and agree to receive sporadic information from Radcal.

If you have any questions or do not wish to be registered, please [get in touch](#).

Enter the Computer ID obtained from the previous step if necessary

-Note the activation key then return to the Accu-Gold 2



End, this is your activation key:

741914

Copy to clipboard

With this key you can enable unlimited measurement in Accu-Gold 2.

Open the Accu-Gold 2 menu and select "Help - Manage Activation". Click on "Activate" and enter your activation key.

You need to register every computer you want to use for measurement. Computers you use for viewing saved sessions don't need to be registered. Registration is free and ensures that we can send you information on important updates or new features. You can see our [privacy policy here](#).

Are you having a problem with the activation? Please get in touch: [Radcal Registration Support](#)

-Enter the activation key the click “Unlock”

1. Start 2. AGDM 3. Registration

x



Get free activation key

Click below to open the online registration form in your browser:

⤴ Offline or having trouble? Activate manually...

Please visit the following link and enter your computer id:

Computer ID:

Registration Link: <https://accu-gold.com/register>

Get Key



Enter activation key

Enter the key you have received online or via email here:



Unlock


We are here for you. Don't hesitate to contact us: cust_sup@radcal.com

-The registration process has now been completed. Click “Close” and enjoy Accu-Gold 2 without restrictions.

1. Start 2. AGDM 3. Registration ×

Success!

Accu-Gold 2 is unlocked for unlimited measurements.



Close

We are here for you. Don't hesitate to contact us: cust_sup@radcal.com

For assistance contact cust_sup@radcal.com