


=====
Radcal, Monrovia, CA (626)357-7921
Software Version 3.30 build 2

For use with Radcal Accu-Gold '+' (plus) series digitizers, Accu-Gold T3 and Accu-Gold Touch Pro instruments in USB mode
=====

To begin Accu-Gold3 software installation, copy the Accu-Gold_Setup_3.x.zip file to your computer. Right-click and select 'Extract All..'. This will expand the zip file and create a folder of the same name. Open the folder

Install new version via setup "**Accu-Gold_Setup_3.30_Build_2.exe**"

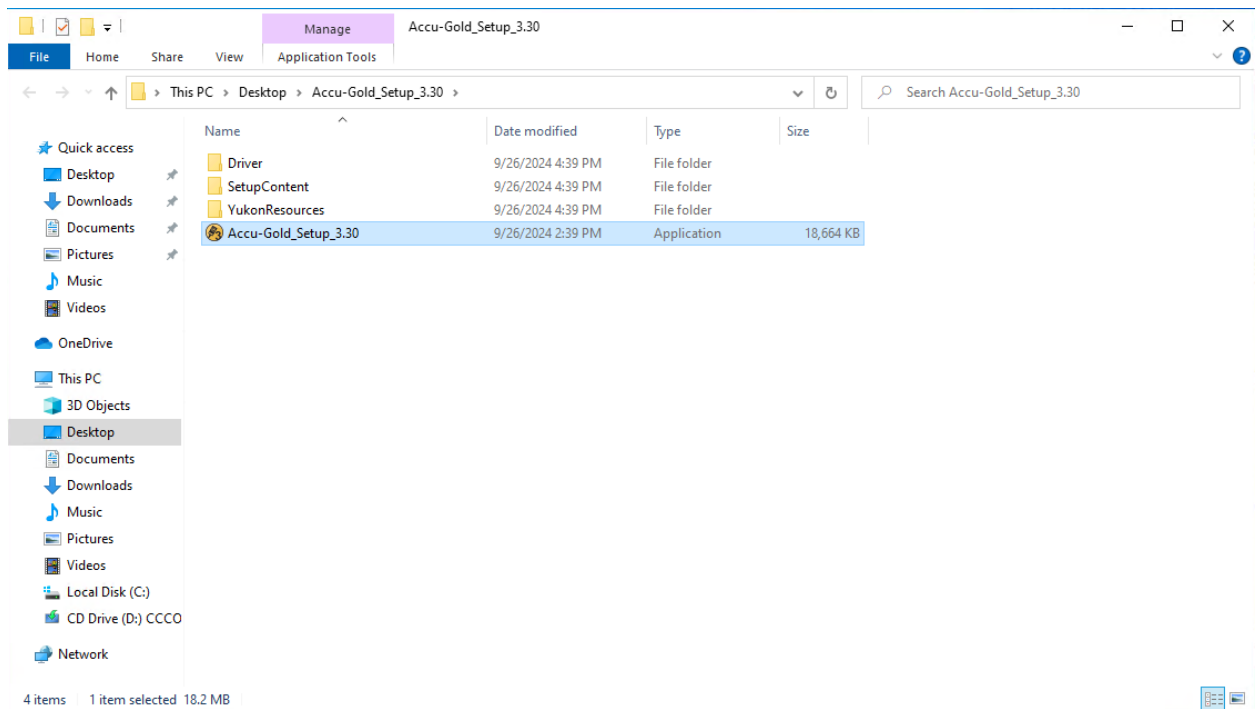

New Feature Summary V3.30

- The installer has been adapted so that it can now be upgraded from any version without any problems and without triggering a possible vulnerability in older installers

Changes

- Changed look and operation of Main Menu
- Added WiFi (T3) connection
- Improved Session management. Sessions now are automatically saved.
- Added ability to add Manual Mode configurations as Favorites
- Measurement display profiles are improved.
- Added measurement profiles. Current library version 5.3.09
- Added this version to website for updating
- Updated 3CT chamber recognition.
- Fixed installation issues on systems with some antivirus software
- Updated User manual
- Update starting cell of Companion Mode