



Release Bulletin

Accu Gold 3 Software & Accu-Gold T3 Firmware

We are pleased to announce the release of **Accu-Gold 3 Software Version 3.35** and **Accu-Gold T3 Firmware 3.9.9**. These updates introduce enhancements and overall system performance. Key updates are listed below.

New Features

- ✓ **Extended Measurement Capability**
Waveform data recorded with the T3 platform or Mobile App can now be imported from AGX sessions and analyzed directly within Accu-Gold 3.
- ✓ **Flexible Dose / DAP / DLP Workflow**
Users can now select their preferred chamber mode (Dose, DAP, or DLP) directly within the workflow, providing greater flexibility for radiography, fluoroscopy, and CT applications.
- ✓ **Improved Pre-Pulse and Combo Mode Visualization**
Pre-pulse regions are now visually highlighted and identified within waveform displays, making analysis of advanced generator pulse sequences easier and more intuitive.
- ✓ **AGX Waveform Import**
Waveform data recorded with the T3 platform or Mobile App can now be imported from AGX sessions and analyzed directly within Accu-Gold 3.

Improved Functionality

- ✓ Enhanced extended measurement processing, including improved handling, visualization, and reporting of long-duration exposures.
- ✓ Improved waveform analysis capabilities with enhanced Combo Mode processing, waveform rendering, and support for complex pulse patterns.
- ✓ Enhanced profile management, synchronization, and compatibility across Accu-Gold 3 and multiple T3 firmware generations.
- ✓ Improved Wi-Fi connectivity, network handling, waveform transfer, and large-file processing.
- ✓ Increased application performance, memory management, and overall system stability through numerous reliability improvements and bug fixes.

Kind regards,



Mr. Daniel Bödeker
Director Product Lifecycle Management
Medical Imaging – IBA Radcal

=====
Radcal, Monrovia, CA (626)357-7921
Software Version 3.35

For use with Radcal Accu-Gold '+' (plus) series digitizers, Accu-Gold T3 and Accu-Gold Touch Pro instruments
in USB mode
=====

AG3 Installation Notes:

1. **Install Accu-Gold software prior to connecting the Digitizer Module.**
Uninstalling old versions is not necessary.
- 2.. Administrator privileges are required for installation.

Installation Procedure:

To begin Accu-Gold3 software installation, copy the Accu-Gold_Setup_3.x.zip file to your computer.

Important: Right-click and select 'Extract All.' This will expand the zip file and create a folder of the same name. Open the folder then double click the setup file.

Uninstall Procedure:

1. To uninstall go to Add/Remove Programs in Control Panel and select "Accu-Gold 3". Then click on Uninstall and answer Yes to the prompts. You may also run the setup program again to uninstall it.