

Accu-Gold T3 Update READ ME-----

V3.9.9 Jul 17, 2026

Upgrading Firmware -

From the Radcal.com website click on Support. Under Downloads click on Accu-Gold Software. On the Accu-Gold Software page and under Accu-Gold T3 Firmware, choose the link [AGT3_UpdateV3.x.x \(Zip download\)](#).

Have a flash drive available and plugged into your computer. Navigate to your Downloads directory and double-click on the zip file **AGT3_UpdateV3.x.x.zip**. It will open in Windows File Explorer. Select all of the files including the folders (Ctrl-A), choose copy, then open up the folder on your flash drive. Choose paste. (Note: it must be FAT32 format, not NTFS)

The T3

When updating please make sure to follow these steps:

- The charged level on the T3 will need to be “medium” or better. If not, installation can proceed but with the charger plugged into the unit.
- Turn on the T3. Wait for it to get to the main menu of initial measurement.
- Plug in the flash drive.
- The update program will run automatically. Do not unplug the flash drive until it has completed.
- When successfully updated, it will say “Update finished...”
- Switch off the T3, remove the flash drive, then turn it back on to start the new program.

If you have any questions, please do not hesitate to contact Customer Support.

<https://helpcenter.iba-dosimetry.com/hc/en-us>

Thank you,
Radcal