

Radcal

Universal Chamber Holder

Universal

Convert your 1800cc or 180cc chamber to a hand held survey meter or mount any Radcal chamber on a tripod



Portability



Convenience



THE GOLD STANDARD IN RADIATION MEASUREMENT

Assembly Instructions

Components

The Universal Chamber Holder consists of 5 components as shown. These components are combined as illustrated in these instructions to produce tripod or hand holding devices that are suitable for all Radcal chambers.



Tripod Mount

Simply screw the support bracket (2) onto a tripod and fit the chamber into the spring clip as shown above. The chamber stem can be rotated in the clip to achieve the desired measurement direction when using "pancake" style chambers such as the 60 and 180cc chambers.

Note: The 60cc chamber requires the 8430 support adapter prior to mounting in the clip. The CT chamber requires the 8430CT adapter.

Hand Support

Fit the handle (5) to the support bracket (2) with screw (3) using the Allen key (4). This converts a Radcal chamber usually 180 or 1800cc into a survey type hand held instrument as shown here.

Note: When making very low level leakage measurements over a prolonged period, it is preferable to support the chamber on a tripod.

Warning: Only make hand held measurements in areas where the radiation dose has been estimated to be less than the permitted personnel exposure level.

